



# EverLift<sup>®</sup>

Submucosal Lifting Agent

**THE LIFT** YOU'VE BEEN WAITING FOR

# EverLift®

## Submucosal Lifting Agent

### INDICATION FOR USE

EverLift Submucosal Lifting Agent is indicated for use in gastrointestinal endoscopic procedures for submucosal lift of polyps, adenomas, early-stage cancers, or other gastrointestinal lesions prior to excision with a snare or other appropriate endoscopic device.

### SUPERIOR CUSHION

#### CELLULOSE BASED & HYPERTONIC

EverLift is cellulose based enabling the solution to stay in place longer, as it is not absorbed as quickly as other lifting agents:



Other Lifting Agents



EverLift

- EverLift is specially formulated with water retaining properties to prevent rapid absorption by the body
- EverLift is hypertonic, and has a slower rate of absorption due to osmotic resistance

EverLift holds a 2mm lift for 60 minutes<sup>1</sup>

1. Data on file

EverLift provides margin delineation with darker color



# THE LIFT

## you've been waiting for



### CONVENIENTLY PACKAGED AS EASY TO USE AS SPOT EX



#### PRE-MIXED WITH METHYLENE BLUE

Methylene blue is used to highlight the lesion perimeter and aids in:

- Increased occurrence of en bloc resection rates
- Reduced occurrence of residual lesions and incomplete resections
- Improved identification of muscle injury to help avoid missed perforations



#### REQUIRES NO PRE-PREP

- EverLift's convenient zero-prep, pre-filled syringe attaches directly to any endoscopic injection needle's luer-lock connection (23G recommended)
- No refrigeration required



### FOR EVERYDAY USE

- EverLift is the only pre-filled submucosal lifting agent on the market available in two presentations, 5 mL and 10 mL
- Use EverLift for all polyps requiring a lift – including smaller polyps (<20mm)
- Use only what you need without throwing out what you've paid for

First Lifting Agent with 5 mL & 10 mL options





For 20 years, GI Supply has been the manufacturer of the leading endoscopic tattoo, Spot Ex, which has been used by physicians and surgeons in more than 5 million cases worldwide.

GI Supply continues to broaden its submucosal injection offering with innovative and affordable devices designed to maximise procedural efficiency and patient safety.



SUPERIOR CUSHION



CONVENIENTLY PACKAGED



FOR EVERYDAY USE

Scan to  
learn more!



## Why choose EverLift?

“This is an effective lifting agent for endoscopic resection that is offered in convenient and cost-effective packaging.”

**Dr. Douglas Rex**

Director of Endoscopy at  
Indiana University Hospital  
in Indianapolis



ORDER NO.	DESCRIPTION	QUANTITY
GIS-55	EverLift Submucosal Lifting Agent 5 mL	Box of 10 Syringes
GIS-59	EverLift Submucosal Lifting Agent 10 mL	Box of 10 Syringes

Disclaimer: EverLift Submucosal Lifting Agent is FDA approved and CE Marked. Please check availability with your local GI Supply distributor.

For More Information Or To Locate A Distributor  
Visit: [www.gi-supply.com](http://www.gi-supply.com)

**GI Supply**  
Specialty Endoscopic Products