

# Disposable Circumcision Anastomat



---

---

**Surgical blade and stapler in one!**

## Product Description:

Anastomat combining surgical blade and stapler.

Developed to cut the foreskin and apply a circular row of staples at the cutting line in 1 step.

## Indication:

Male circumcision (Paediatric and Adult)

Suitable for patients with phimosis and redundant prepuce

## Sizes:

8 sizes available (Contact us for sizing chart and order codes )

A sizing tool is provided for selection of correct stapler size



### **Large visual window**


Allows visibility of marking line.

Position of prepuce can be adjusted at any time.



### **Concave seat & frenulum observation window**

Inner concave seat designed to be placed inside the prepuce to cover and protect the inner glans.

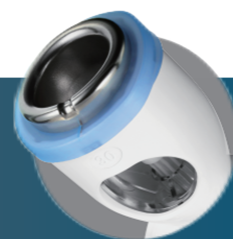

### **Frenulum Protection Groove**

Frenulum should be aligned with the protective groove in the concave seat.  
Positioning of the frenulum can be adjusted through the observation window.



### **Safety Clip**

Safety clip ensures staples don't fire prematurely.

### **Automatic Handle Lock**

The automatic lock secures the handles in place to trigger cutting and deployment of staples.

